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[EN] Consumer Information

Strata-triz is a rapidly drying, non-sticky, transparent, silicone gel formulation for the treatment of scars, both old and new.

Strata-triz is easily applied to all areas of the skin, including exposed areas like the face and neck as well as joints and hairy areas without the need for shaving. When used as directed Strata-triz dries to form a silicone gel sheet, a protective layer that is gas permeable and waterproof which hydrates and protects your scar from chemical and microbial invasion, allowing scars to normalize the collagen synthesis cycles, producing a normal mature scar.

Strata-triz is used to soften and flatten scars, relieve the itching and discomfort of scars as well as reduce redness and discoloration associated with scars.

Strata-triz was developed for use on all types of scars including those resulting from: general surgery, trauma, chronic wounds, burns, bites, acne and other diseases of the skin.

The 2002 International Clinical Recommendations on Scar Management stated... "silicone gel should be used as first line therapy in the initial management of scars and in the prevention of Hypertrophic scars and Keloids."*

Intended use: Strata-triz is used for the prevention and treatment of Keloids and Hypertrophic scars. Strata-triz can also be used in conjunction with other invasive scar management options to improve overall results, including surgical excision and resuturing, intralesional injections and pressure garments. Strata-triz is suitable for children or people with sensitive skin. Strata-triz is safe to use on infants, during pregnancy and whilst breastfeeding.

Directions for use: Ensure that the scar is clean and dry. Apply a very thin layer of Strata-triz to the scar and allow to dry. Strata-triz should be applied once daily, or twice daily to exposed areas or as advised by your physician. For best results Strata-triz should be maintained with in continuous contact with the skin (24 hrs day). When applied correctly, Strata-triz should be dry in 3–4 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a tissue and allow the drying process to continue. Once dry, Strata-triz can be covered by sunscreen, cosmetics, pressure garments or casts.

Recommended duration of treatment

Strata-triz is recommended as minimum treatment of 60–90 days (24/7). Larger and older scars may take longer. Continued use is recommended until no further improvement is seen.

How much Strata-triz do I need?

- Size 5 g (0.17oz) is enough to treat a 2–3 cm (1–2 inch) scar for over 90 days.
- Size 10 g (0.35 oz) is enough to treat a 5–7 cm (2–3 inch) scar for over 90 days.
- Size 20 g (0.7 oz) is enough to treat a 10–12 cm (4–5 inch) scar for over 90 days.
- Size 50 g (1.75 oz) is enough to treat an area of 6 × 12 cm, (2.5 × 5 inch) for over 90 days.

Caution

- Strata-triz should not be applied on third degree burns or to open wounds.
- Strata-triz should not be placed in contact with mucous membranes, or the eyes.
- Strata-triz should not be applied over other skin treatments without the advice of your physician.
- Strata-triz may stain clothing if not completely dry.

For external use only. If irritation occurs, discontinue use and consult your physician. Keep this product out of the reach of children.

Store below 25°C (77°F). Do not use after the expiration (EXP) date printed on the tube.

Ingredients: Polydimethylsiloxanes, Siloxanes, Alkylmethyl Silicones

Stratapharma AG, Centralbahnplatz 8,
CH-4051 Basel, Suiza/Suïça/Schweiz/Switzerland

Fecha de preparación de esta información/
Data de preparação desta informação/
Erstellungsdatum der Informationen/
Date of preparation of this information: 12/2014

*Mustoe TA et al. Plast Reconstr Surg 2002;
110:560–571

www.strata-triz.com

Stratapharma
Switzerland

ST000ESPPI01-1214